

Sereneview Cubicle Curtain Care & Maintenance

With the proper care and maintenance, Sereneview's Avalon fabric with Armor Protection can last over 10 years and our mesh 2-5 years. Excessive heat will impact the mesh much faster than the fabric. A dedicated curtain/drapery cleaning service will maintain the appearance of the curtains and extend their useful life.

Wet Cleaning: Use mild soap or detergent in water NOT TO EXCEED 160°F (synthetic setting) with a high water level and mild agitation. Extract Gently. **DO NOT USE BLEACH.** Repeated laundering with bleach will remove all color from the fabric over time. Clorox Healthcare Peroxide Cleaner Disinfectant is 99.9% as effective as bleach and can be used in the laundry. Applying bleach directly to the fabric will adversely effect the color. Clorox Broad Spectrum Quaternary Disinfectant Cleaner can be used as a spray on hard and soft surfaces. Another excellent disinfectant and bleach alternative is FabricAide. Do NOT use fabric softeners.

Spot Cleaning: This fabric has been processed with Synpel A, a bodily fluid and stain repellent. Stains can be spot cleaned with a damp sponge or cloth and a typical NON-BLEACH cleaner/germicidal including; oxidizing agents, alcohol disinfectants, quaternary disinfectants and industrial laundry detergents. The repellent properties have a useful life of twenty-five (25) commercial launderings.

Dry Cleaning: Do not dry clean.

Drying: Air, cabinet, or tumble dry at a low temperature. Do not exceed 3 to 5 minutes on synthetic cycle at a maximum of 104°F. Hang or fold immediately after drying.

Finishing: If necessary, press as follows: Hand Iron - Iron at a low heat setting (275°F) using a damp cloth or Steam Iron with a dry press cloth. Grid Head Press - Press for short intervals with minimum steam. Do not lock head. Flat Bed Press - Press the dampened drapery using a cloth covering.

Antimicrobial: Avalon fabric has Aegis Microbial Shield® technology (3-(Trimethoxysilyl) propyldimethyloctadecyl ammonium chloride) applied during manufacturing and does not require additional antimicrobial treatment after laundering. The technology is durable and non-leaching. For more information about Aegis Microbe Shield technology, go to: www.aegisasia.com.

UV Disinfectant: Avalon fabric has been tested with an ultra-violet disinfecting system and achieved better than 70% reduction of colonies without any impact on the fabric while meeting the manufacturer's specifications.

Safety Compliance

All fabric used for Sereneview curtains is inherently/permanently flame-retardant and passes both NFPA 701 and California Title 19 after seven (7) commercial launderings. Additionally, the fabric has an antimicrobial treatment that does not leach over time and resists MRSA, C. Diff, mold, various Salmonella strains, and a wide array of other microbes. More information is available on our web site: www.sereneview.com. Reports of both are available on request.

Mesh: Curtains with mesh have 1/2" nylon mesh with a minimum standard width of 20". The mesh width can be increased up to 36" to accommodate different ceiling heights. Anything with less than 20" of clearance may not pass a fire inspection. Consult your local fire marshal for more information.

Location: To find out the location of the photos go to: www.sereneview.com/image_library.php and click the curtain width of your curtain and then Style, scroll down to view the photos. The location is below the title.